EN – FFP3 NR – Single Use Particle Filtering Half Mask

User Instruction

Dear Customer,
Thank you for purchasing this Trend product, which should give lasting performance if used in accordance with these instructions.

EN149: 2001 standard:

Complies with the requirements of the Personal Protective Equipment Regulations (Regulation (EU) 2016/425 by means of the European Standard EN149:2001+A1:2009, as brought into UK law and amended). Three protection classification – Filter efficiency against particulate aerosols is greater than 80%(FFP1), 94%(FFP2), 99% (FFP3). Classification is showed on the mask.

Product Marking explanation

<p>trend → Identification of manufacturer RPE/FFP3V → Product model name EN149:2001+A1:2009 → Standard</p>	<p>FFP3 → Protection classification NR → Single shift use only CE → CE Logo 2797 → Notified Body UK CA → UKCA logo 0086 → Notified Body</p>

Check prior to use:

- Careful observance of these instructions is an important step in safe use.
- Check the face seal is welded well.
 - Check the nose piece is glued closely on the mask.
 - Check the valve is set up well.
 - Check the valid date which is showed on the box.

Fitting instructions:

- Cup the mask in your hand with the nose piece at your fingertips allowing the headbands to hang freely below your hand.
- Cup the mask firmly against your face with the nose piece on the bridge of your nose.
- Pull the bottom strap over your head and position it around the neck below the ears. Pull the top strap over your head resting it high at the top back of your head.
- RPE/FFP3V has an adjustable strap buckle. Adjust tension by pulling head-straps with both hands. Decrease it by pushing out on the back of the buckle.
- Check the face seal is fully fitted your face. Press soft nose piece to conform snugly around the nose.
- (a) To test fit for mask without exhalation valve: Cup both hands over the mask. Then inhale and exhale vigorously, (b) To test fit for mask with exhalation valve: Cup both hands over the mask and inhale sharply.
- When inhaling, a negative pressure should be felt inside mask. If any leakage is detected, adjust position of mask and/or tension of strap. Retest the seal, and repeat the procedure until the mask is sealed properly.

WARNING:

- Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of the mask and result in sickness or death.
- A properly selected mask is essential to protect your health. Before using this mask, consult an industrial hygienist or Occupational Safety Professional to determine the suitability for your intended use.
- This mask does not supply oxygen and must not be used in atmospheres containing less than 19.5% oxygen.
- Do not use when concentrations of contaminants are immediately dangerous to health or life.
- Do not use the mask in an explosive environment.
- Natural rubber strap may cause allergic to certain people, advise the user to stop using the mask if this happen

- Leave work area immediately and return to fresh air if breathing becomes difficult, or other distress occurs.
- Do not use with beards or other facial hair that may inhibit contact between the face and the edge of the mask.
- Never alter, modify or abuse this mask.
- Do not use for more than one shift (8hours). No maintenance necessary. Discard the used mask after use.
- Change this mask immediately if breathing becomes difficult or the mask becomes damaged or distorted, or a proper face fit can't be maintained.
- Store unused masks in box in a clean, dry, non-contaminated area.

<p>Date of expiry: 3 years</p>	
<p>CE → 2797</p>	
<p>UK CA → 0086</p>	
<p>EN149:2001+A1:2009</p>	

FFP3 NR
RPE/FFP3V

Notified Body: 2797
BSI Group The Netherlands B.V.
Address: Say Building, John M. Keynesplein 9, 1066 EP, Amsterdam, Netherlands

Notified Body: 0086
BSI
Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes, MK5 8PP

Manufacturer
Trend Tool Technology Ltd.
Watford, WD24 7TR, England
www.trend-uk.com

You can find the EU & UKCA declaration of conformity at:
www.trend-uk.com/declarations-of-conformity

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> 5 027654 073582

www.trend-uk.com
 technical@trend-uk.com
 Tel. 0044(0)1923 249911
 3rd Floor, Kilmore House,
 Watford, WD24 7TR, England
Trend Tool Technology Ltd.
EU Importers
Trend Tool Technology Ltd.
 @ All trademarks acknowledged E&OE
 © 2022 Trend Tool Technology Ltd.

ENVIRONMENTAL PROTECTION
 Knowledge, Innovation, Quality
 BSI Assurance UK Ltd, MK8 8PP, Nottingham Body 0086
 Netherlands, Notified Body Number CE 2797
 John M. Keynesplein 9, 1066 EP, Amsterdam, The
 BSI Netherlands, company registration number 33264284,
 recycling.
 The product and its accessories at the end of its life should
 be sorted for environmental-friendly recycling.
GUARANTEE
 All Trend products are guaranteed against any defects in
 either workmanship or material, except products that have
 been damaged due to improper use or maintenance.

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